

SOF[®] TOURNIQUET

Part#: 84-0009 | NSN: 6515-01-696-4522

US PATENT-8,303,620 and D649,642 and 7,776,064

INSTRUCTIONS FOR USE


To Apply the Tourniquet



**Slide Over Limb
Above the Injury**



Pull Tail Tight

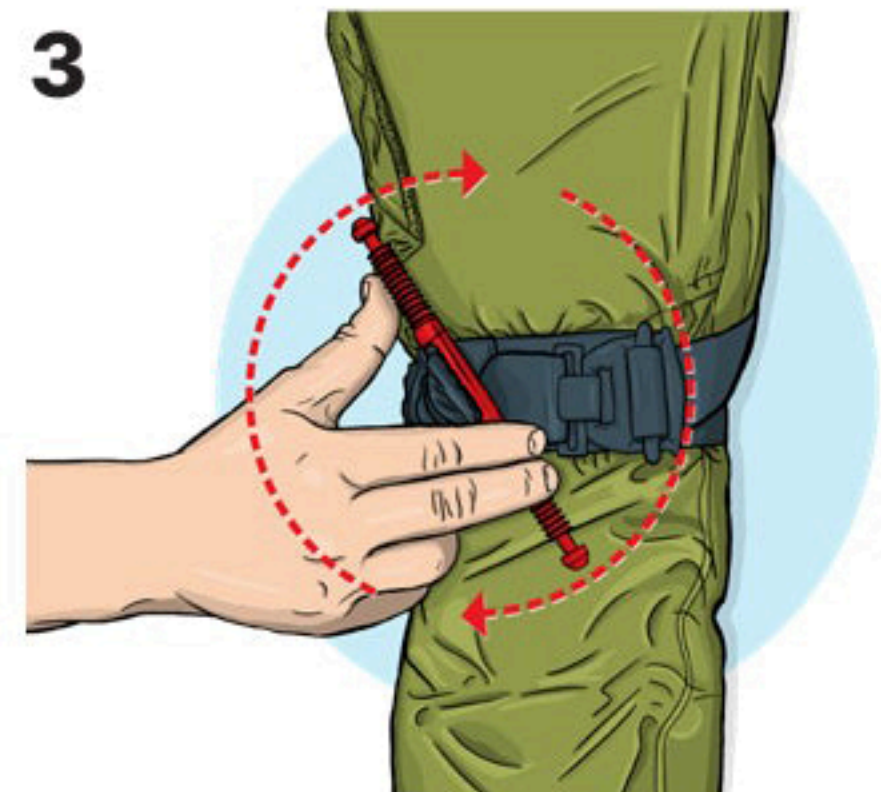

**Twist Handle
Until Bleeding Stops**




**Optional: Temporarily
Brace in Clip**




**Lock and Secure
in Triangle**



Note Time

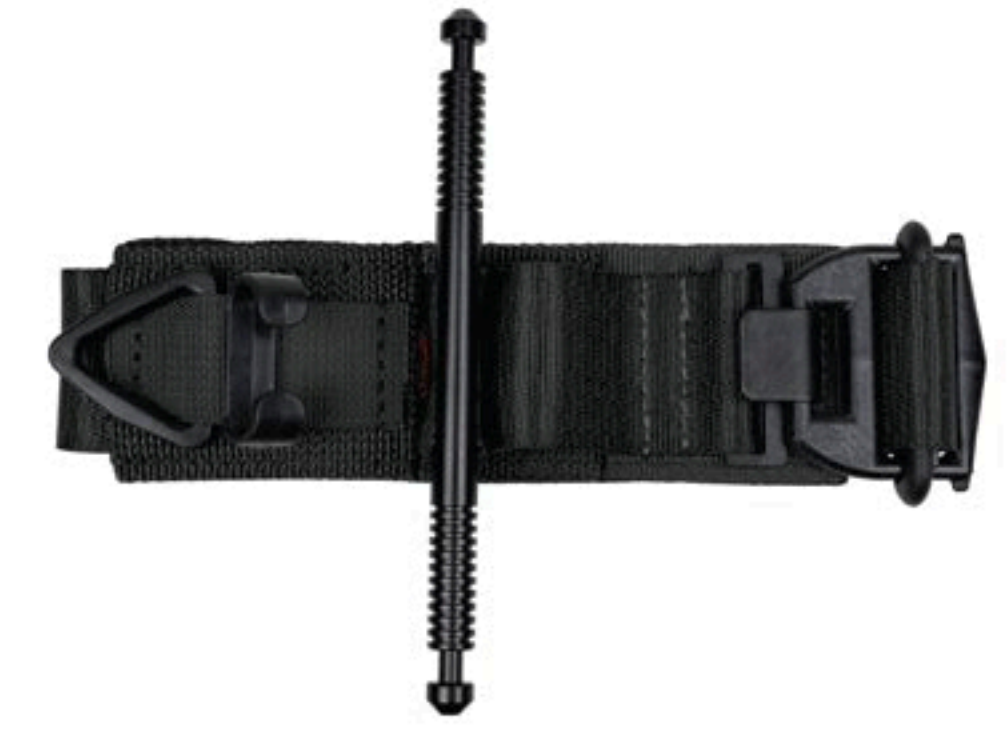


Applying the Tourniquet if Unable to Slide Over Limb Follow steps 1b-4b

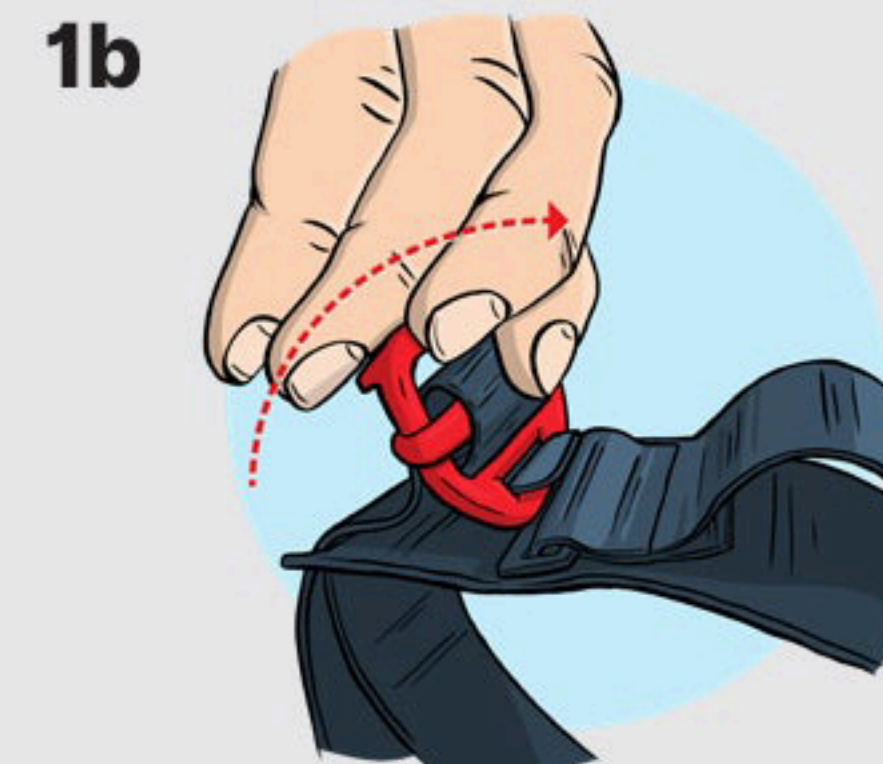


Life Buckle



Twist to Disconnect

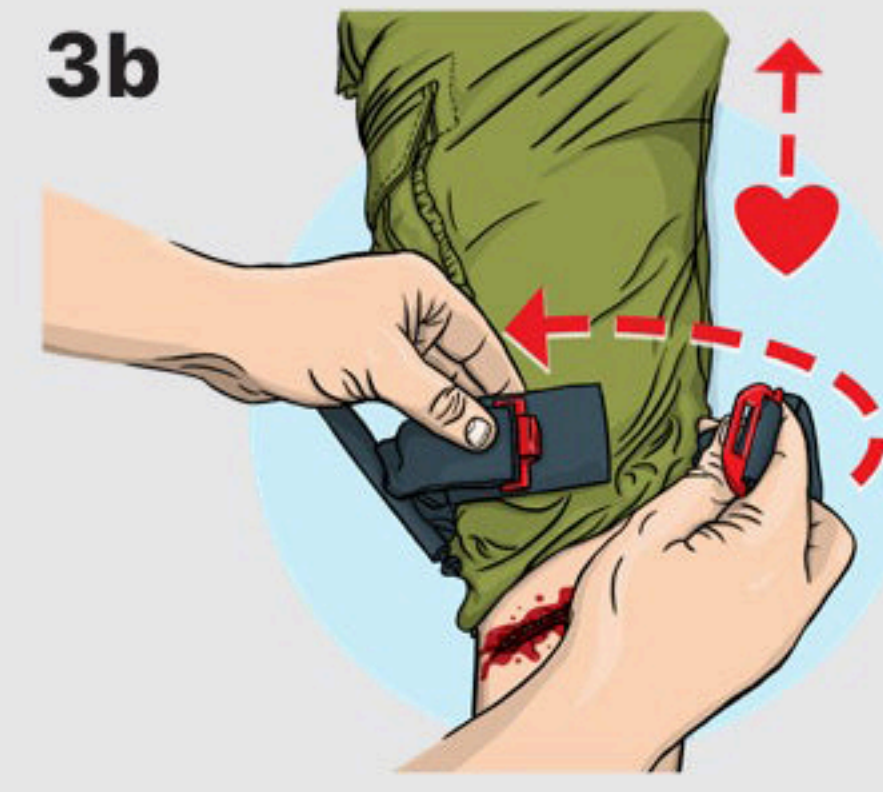

Go Around Limb

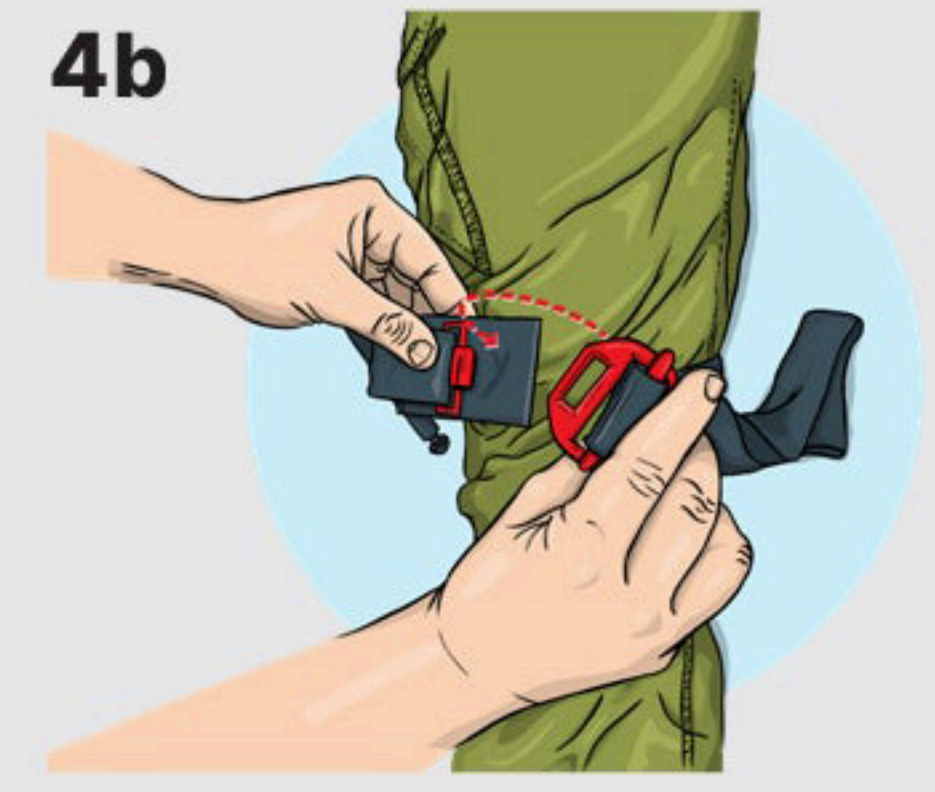

**Connect Buckle, Then
Go to Step 2**

SOF[®] Tourniquet
For control of severe bleeding.

SOF[®] Tourniquet Storage
Store the tourniquet with the buckle connected and
enough slack to slip the tourniquet loop over your limb.

**WARNING:
FOR SINGLE
USE ONLY**



TACMED SOLUTIONS[™], LLC

PHONE 864.224.0081
TOLL FREE 1.888.822.6331
FAX 864.224.0064
E-MAIL info@tacmedsolutions.com
WEB tacmedsolutions.com
1250 Harris Bridge Road, Anderson, SC 29621, USA

